

# Storz C-Mac

## Connecting the C-Mac

- 1) Locate the Storz connection box and the cable in the room. This should be located on the anesthesia machine to the left of the vitals monitor.



- 2) Align the notches on the male cable connector to the port on the handle of the C-Mac blade.



- 3) Plug in the connector into the C-Mac blade and the blue light on the handle as well as the main light on the blade should automatically turn on.




## Projecting the Image onto Room Monitors


- 1) Locate Stryker control panel on the boom behind the anesthetic machine.



- 2) Select the "Route Video" button on the bottom left corner of the screen.

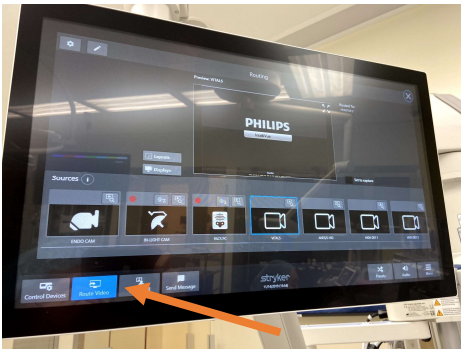

- 3) Select "ANES/S-VID" button.

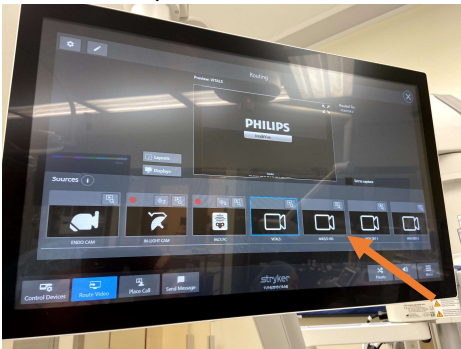

- 4) Select the monitor you wish to use for the C-Mac image. Monitors should be appropriately labelled in each room. (Monitor 3 is selected in this example.)

