


LTV[®] 1200 ventilator Emergency Setup Card

For 24-hour assistance 800.754.1914



Step 1: Choose a power source


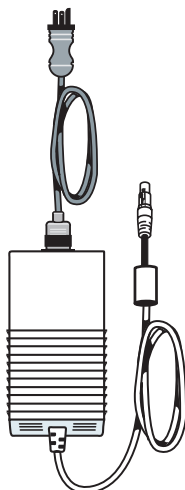


AC adapter
(use AC power when available)




SprintPack Lithium-Ion
Power System

Step 2: Connect LTV® ventilator to power source

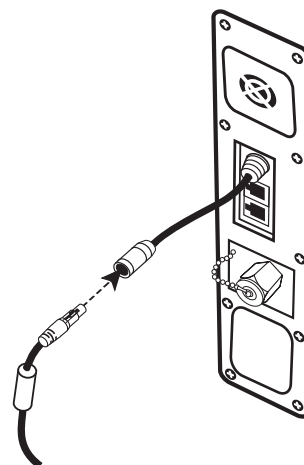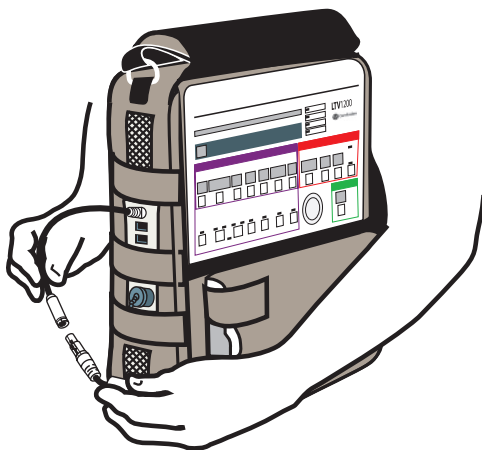

Step 3: Choose a circuit

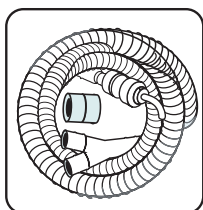

- Open bag
- Remove patient circuit and 22 mm adapter from bag

Humidifier circuit (optional)




22 mm patient circuit, **adult**

>20 kg

>44 lbs

22 mm adapter

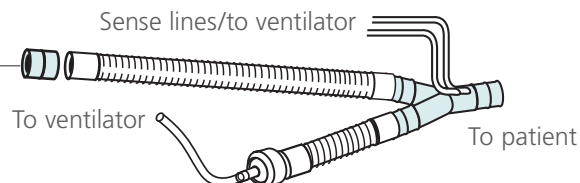


15 mm patient circuit, **infant/pediatric**

5kg to ≤ 20kg


11lbs ≤ 44lbs

22 mm adapter

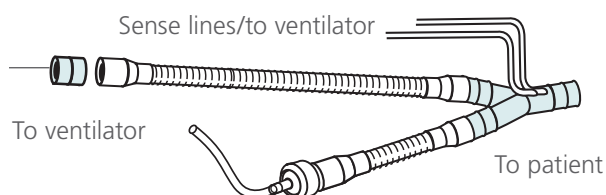


Step 4:

Connect circuit to LTV ventilator

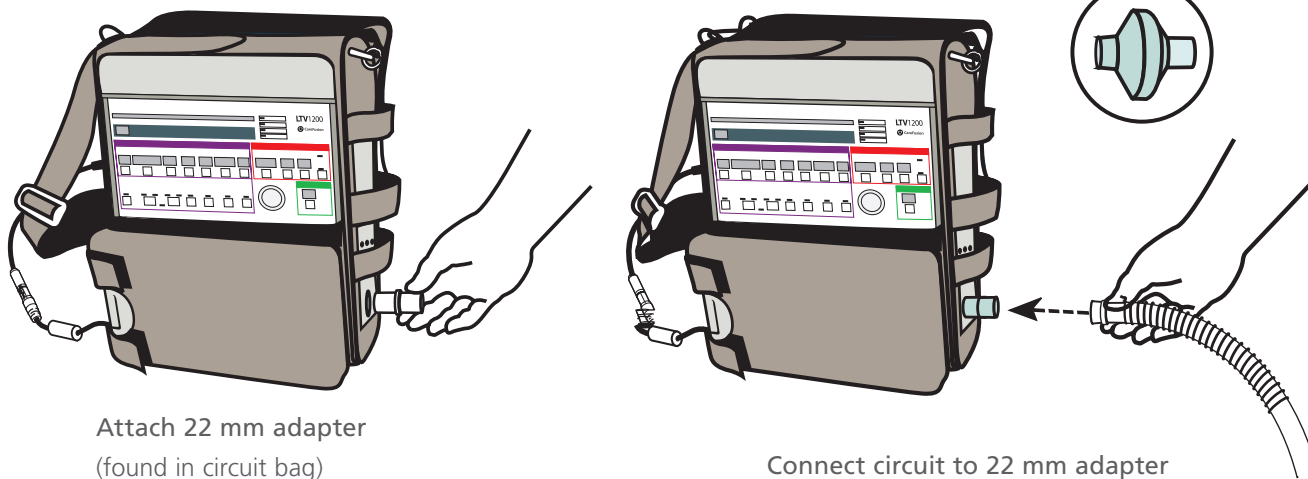


Step 5:

Connect sense lines to LTV ventilator

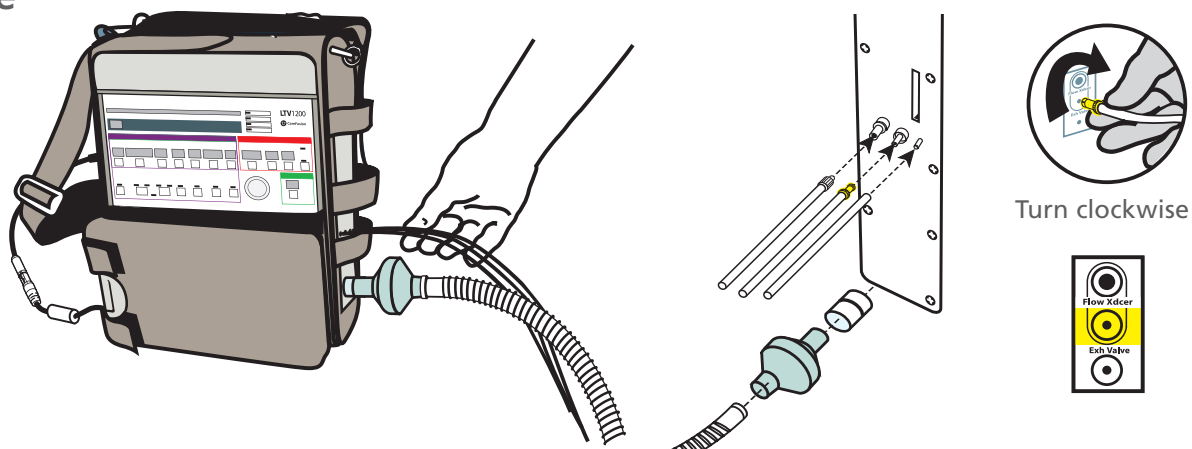


Step 6:

Turn LTV ventilator on

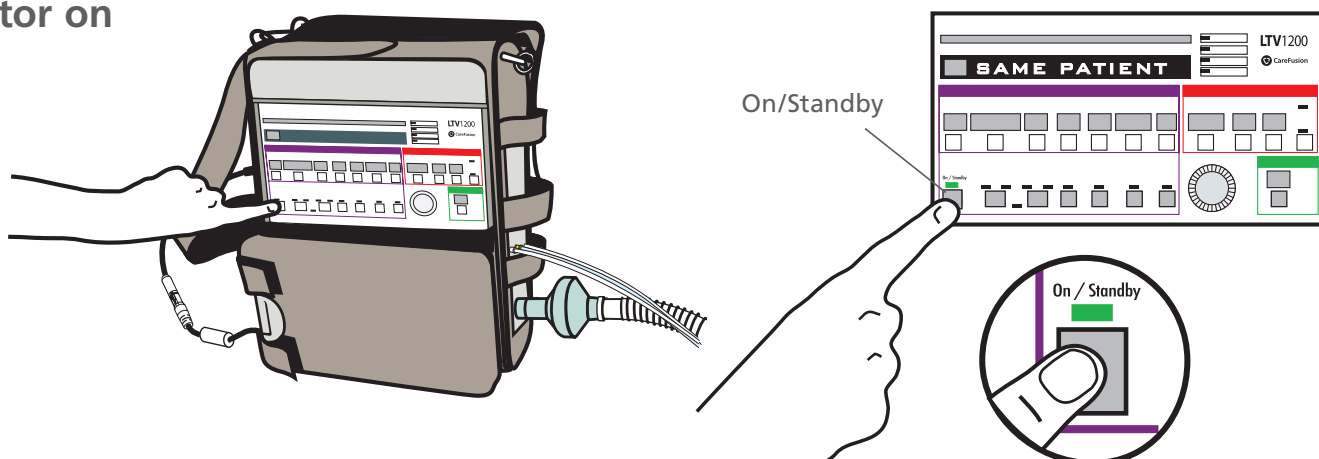


Step 7: Patient setting

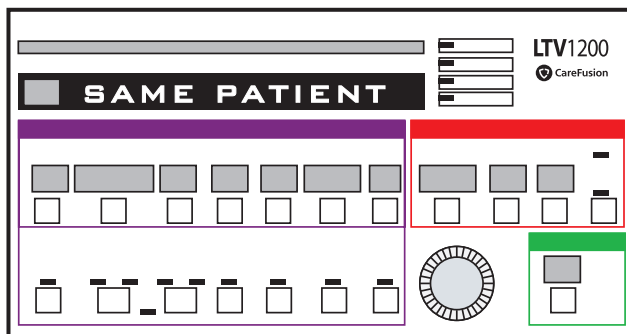


Once power is on, see **Same Patient** in display

Step 8: New patient setting

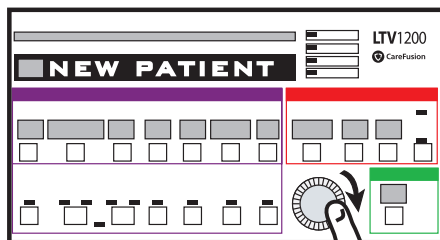


Rotate **Set Value** knob clockwise until **New Patient** is displayed

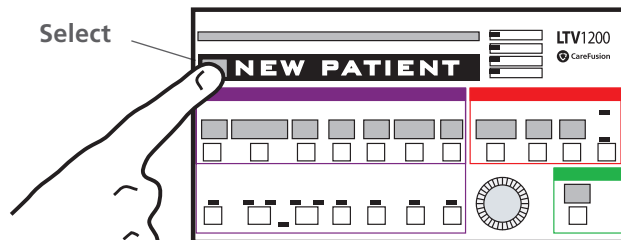


Press **Select** when **New Patient** is displayed

Step 9: Set for patient size

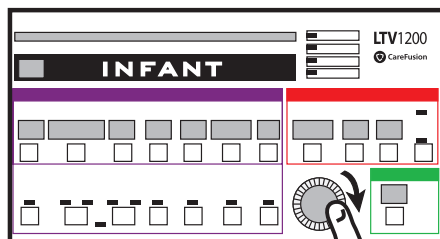

Rotate **Set Value** knob to correct patient size (infant, pediatric or adult)

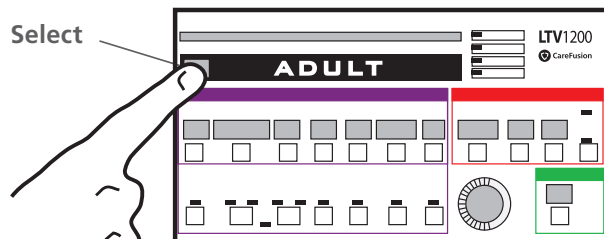


Press **Select** when proper patient size is displayed

MISTAKE: If you make a mistake, turn the LTV ventilator off by holding the On/Standby button for 3 seconds. Then return to Step 6.

Step 10:


Connect circuit to
ET-tube or mask

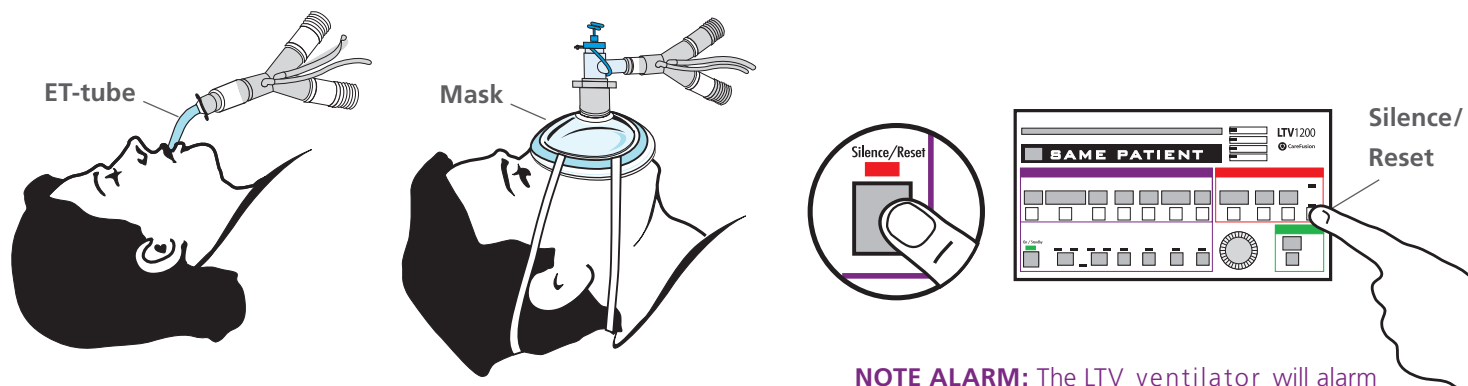

NOTE ALARM: The LTV ventilator will alarm until it is connected to the patient. You may press the **Silence/Reset** button.


Step 11:

Optional/using
a mask

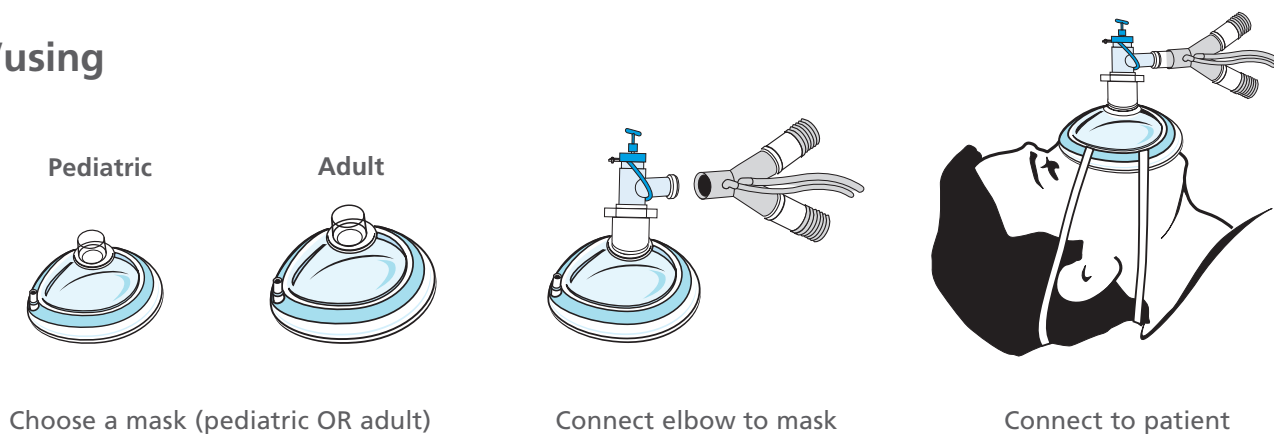



Step 12:

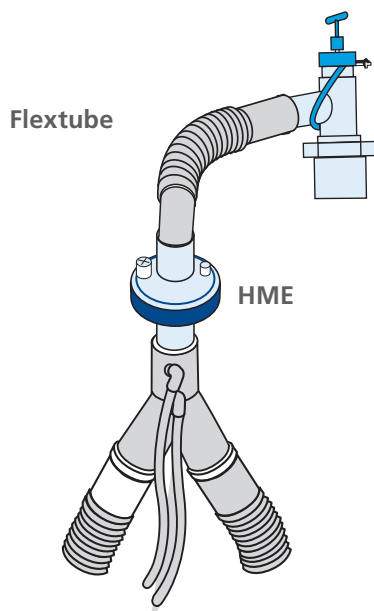Optional/using
an HME and/
or flextube
as shown
for patient
comfort



This emergency setup card is intended to provide emergency instructions for initial setup of the LTV 1200 ventilator. The initial settings may need to be adjusted based on the patient's clinical condition. For complete information regarding setup and operation of the ventilator, refer to the LTV 1200 ventilator operator's manual.

 CareFusion
22745 Savi Ranch Parkway
Yorba Linda, CA 92887
800.231.2466 toll-free
714.283.2228 tel
714.283.8493 fax

 CareFusion Germany 234 GmbH
Leibnizstrasse 7
97204 Hoechberg
Germany
+49 931 4972-0 tel
+49 931 4972-423 fax

CE
0086

CareFusion
Yorba Linda, CA

prepare.carefusion.com



© 2010 CareFusion Corporation or one of its subsidiaries. All rights reserved. LTV is a trademark or registered trademark of CareFusion Corporation or one of its subsidiaries. RC1304 (1210/2000) P/N 18354-001 Rev. D