

SHROUDED CAPR AIRWAY TEAM DONNING AIRWAY PPE

SLOW DOWN, SLOW IS SAFE!

DONNING AREA INSTRUCTIONS WITH DOUBLE SHROUDED CAPR

1. IDENTIFY SAFETY OFFICER/BUDDY
2. REMOVE ALL PERSONAL ITEMS. NOTHING COMES OUT THAT GOES IN!
3. GET ENHANCED PPE AND AIRWAY BOXES, CAGED CAPR, 2ND SCRUB PAIR (1 size up), BOUFFANT, AND YELLOW GOWN, THEN ENTER ANTECHAMBER AND PLACE ON THE TABLE
4. HAND HYGIENE/GEL
5. PUT VA PROVIDED SCRUBS ON TOP OF WORN CLOTHING (optional).
6. HAND HYGIENE/GEL
7. Optional: surgical mask or N95 if desired
8. BOUFFANT CAP/BEARD COVER.
9. SIT DOWN AND PUT ON ORTHO BOOTIES.
10. HAND HYGIENE/GEL
11. STERILE GLOVES 1ST SET
12. YELLOW GOWN* do not tie yet*.
13. PLUG CAPR HELMET CORD INTO BATTERY AND CHECK INDICATOR LIGHTS ON FRONT OF HELMET – NEED AT LEAST 2 GREEN LIGHTS TO ENTER HOT ZONE

14. ATTACH CAPR BATTERY TO BELT THEN CLASP BUCKLE AROUND WAIST
15. ROTATE BATTERY BEHIND YOUR RIGHT HIP
16. WHILE HOLDING HELMET - ATTACH CAPR DOUBLE SHROUD HOOD TO HELMET: FRONT HOLE ONTO FRONT HOOK, 2 SIDE HOLES TO 2 SIDE PEGS
17. PULL HOOD OVER HELMET: GRIP BACK SIDE OF HOODS PULL FROM FRONT TO $\frac{3}{4}$ OF WAY OVER TOP OF CAGE HELMET. GENTLY PULL HOOD OFF OF VELCRO IF NEEDED. Pull hood over $\frac{3}{4}$ of way over top of cage helmet grabbing from back part of filtration material, gently pull filter material off front Velcro if stuck there
18. Rotate CAPR helmet knob counter clockwise to open it fully
19. PLACE HELMET CORD OVER RIGHT SHOULDER, THEN DON HELMET, START AT REAR OF HEAD
20. HELMET CHECK – ENSURE HELMET IS STRAIGHT: VELCRO STRIP SHOULD BE $\frac{1}{2}$ INCH OVER EYEBROWS, INDICATOR LIGHTS IN PERIPHERAL VISION
21. ADJUST **HELMET FIT** by dialing knob until snug
22. CHECK **HELMET FIT** by leaning forward then moving head side to side
23. **TIE YELLOW GOWN**

24. Pull CAPR filtration hood down, gently tug at rear while holding under lens in front to pull it down. Two layers should be smooth
25. Take heavy loading filter cap (MaxAir logo in the front)
26. Start with rounded rear, attach heavy loading filter cap on top of filtration hood, end by attaching the front (logo) end to the Velcro strip
27. Pull Outer shroud up, away from inner shroud
28. **SURGICAL GOWN ON** – TUCK INNER SHROUD INSIDE SURGICAL GOWN, LEAVE OUTER SHROUD OUTSIDE OF SURGICAL GOWN, then secure neck Velcro, tie back and sides
29. Pull neck ties gently from back of neck and tie leaving 1" space around neck. Pull waist ties gently, lace through front slits near your waist, and tie firmly around torso
30. **2ND PAIR OF STERILE GLOVES ON. FIT OVER SURGICAL GOWN SLEEVES.**
31. **DOUBLE CHECK HEAD TO TOE COVERAGE.**
32. **BRING AIRWAY SUPPLIES, GLIDESCOPE, WALKIE TALKIES, VENTILATOR AND DRUG BAG INTO HOT ZONE**

CONTACT CARLOS OR JODY FOR CHANGES